


INSTRUCTIONS FOR USE

1. To expose plunger remove large cap
2. To expose needle, pull colored cap straight off being careful not to bend needle.
3. Use as directed by physician. Measure correct dose by lining up the edge of the plunger tip with the correct barrel. To minimize air bubbles, draw your insulin slowly.
4. After use, dispose of the syringe in accordance with local safety standards to prevent misuse

WARNING: For patients who mix two types of U-100 insulin. Do not change (a) the order of mixing that the physician has prescribed or (b) the model and brand of the syringe or needle without first consulting your physician or pharmacist. Failure to heed this warning can result in dosage error. Discard used syringe in sharps container in accordance with facility, local, state, and national regulations. Do not use if protective cap is missing.

CAUTION: Do not resheat used needles. Dispose of used needles and syringes in a sharps container. Do not store at extreme temperatures or humidity.



 (01)00810106252872

 (17)00101

 (10)yyyymmxxx

MedZonHealth*

REF: MZH-VT-118
 LOT: YYYMMxxx
 YYY-MM-DD


EasyCare
INSULIN SYRINGE WITH NEEDLE
0.3mL 31Gx⁵/₁₆"



NON-TOXIC NON-PYROGENIC



- Ultra-sharp needle tip minimizes patient discomfort
- Cartridge slides to lock needle into safety position
- Safety feature ensures protection against needle stick injuries
- Double-wing design
- Sterile and individually packaged
- Single-use only
- Latex-free, DEHP-free

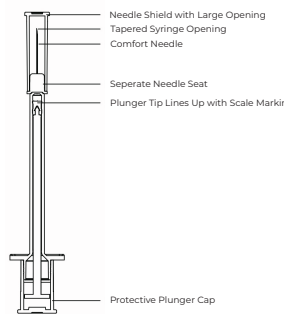


FOR USE WITH U-100 INSULIN ONLY

10PCS YOUR PARTNER IN MEDICAL SUPPLIES



Rx Only  

STERILE EO  

STERILE   

Manufactured For:
MedZonHealth*
 An Abbadi Holding Company
 120 Newport Center Dr. Newport Beach, CA 92660
 Tel: +1 949 909-0253
 Fax: +1 714-398-8188

Production Notes:
 DI = (01)00810106252872
 PI = (17) To be entered during production
 PI = (10) To be entered during production
 GS1 DataMatrix 2D barcode is generated from this data.